

TECHNICAL DATASHEET FOR ORSAM FIXED HEIGHT TRANSPORT TROLLEY

Reference

| | | |
|--------|--------|--------|
| 513001 | 513002 | 513003 |
| 513004 | 513005 | 513006 |
| 513007 | 513008 | |



Designation

Fixed Height Transport Trolley for Steam Sterilizers

Description

Trolley for moving loading cart of steam sterilizer in or out of chamber. It is also used to move loading trolley to necessary area.

Usage

ORSAM Fixed Height Transport Trolleys are used with **ORSAM** steam sterilizer which have loading carts. Duty of transport trolley is to deliver loading cart from dedicated area to the sterilizer. It is also used to load the loading cart inside of chamber and unload too.

Quality Certification & Compliance

Fixed Height Transport Trolleys are not subject to CE where a declaration of conformity may be provided upon request. Design criteria of trolleys are according to 89/392/EEC.

Materials Used & Main Features

Fixed height transport trolleys are made of AISI 304 quality stainless steel. TIG welding is used for connections when necessary or otherwise is not possible. All sharp sides are rounded.

Four castors of 100 mm diameter are used for mobility.

Two of the castors are with brakes in order to prevent unwanted movements after docking to sterilizer especially. Each castor can be height adjusted manually with nuts to prevent problems

✉ Kirazlık Mah. 1029.Sk.No:33 Tekkeköy/Samsun/ TURKEY

☎ +90 362 275 1 168 / ☎ +90 553 311 73 07

✉ info@orsamtip.com.tr / sales@orsamtip.com.tr


www.orsamtip.com.tr



in unbalanced floors. Each transport trolley has a locking system which can be easily operated by user to fix the loading cart to trolley. That is used to prevent unwanted movements and accidents. Movable rod locks the loading cart located on top of transport trolley when it is on the right. When transport trolley is placed to the dock inside the chamber move the rod to the left and push the rod forward free the loading cart movement and free the transport trolley from the dock inside chamber respectively. Top level of transport trolley is with a railing system in order to provide easy movement for loading cart. Height of the top level of transport trolley can be adjusted with nuts in order to align height. All transport trolleys are designed to meet the loading criteria of a specific **ORSAM** model.


Available Types

| Reference No | Suitable Models for Trolley | Notes |
|--------------|-----------------------------|---|
| 513001 | ORSAM 1S & ORSAM 1M | Same frame is used with different loading cart docking station length |
| 513002 | ORSAM 1L & ORSAM 1XL | Same frame is used with different loading cart docking station length |
| 513003 | ORSAM 2 & ORSAM 4 | Same frame is used with different loading cart docking station length |
| 513004 | ORSAM 6 | - |
| 513005 | ORSAM 8 | - |
| 513006 | ORSAM 10 | - |
| 513007 | ORSAM 12 | - |
| 513008 | ORSAM 14 | - |

Please request information from your Agent / Sales Representative for getting more information on tailor made solutions of **ORSAM** series.

 Kirazlık Mah. 1029.Sk.No:33 Tekkeköy/Samsun/ TURKEY

 +90 362 275 1 168 /  +90 553 311 73 07

 info@orsamtip.com.tr / sales@orsamtip.com.tr

www.orsamtip.com.tr

Cleaning

Clean the unit with a dry fabric. Use only stainless-steel maintenance spray for upgraded level of protection.


Comments



The product must be used by specialized personnel only.

Check the package's integrity before use.

Do not use if the package is open or damaged.

Our company has various designs and several technical variants. There may be differences between the same product types and various versions.

 Kirazlık Mah. 1029.Sk.No:33 Tekkeköy/Samsun/ TURKEY

 +90 362 275 1 168 /  +90 553 311 73 07

 info@orsamtip.com.tr / sales@orsamtip.com.tr

www.orsamtip.com.tr